

SIMBABY Start-up instructions

To charge:

1. Connect the black SimBaby powercord to SimBaby, and connect the powercord to the power cable. (Ensure it's the correct charger for the SimBaby, NOT the charger for the SimPad. Refer to label on charger. Make sure SimPad also charged)
2. Should see a flashing Orange light on SimBaby to indicate it's charging.

SimBaby using Simpad start-up

1. Power on the Simbaby by pressing the power button. Light will be green. Baby will cough to indicate he is on.
2. Power up SimBaby SimPad
3. Choose Manual Mode
4. Select " Healthy patient"
5. Start
6. To power down, press the on/off button. Select the "Sim pad/Simulator" option and shut down.

SimBaby using LLeap

1. Power on SimBaby as above. DO NOT power up SimPad.
2. Turn on Instructor PC laptop and log on as per your usual log on.
3. Launch LLeap programme
4. Ensure you are connected to WiFi
5. If not, Click on Network Selector, find SimLink network and press OK when prompted
 - o Prompted to allow, select "yes",
 - o then the Network should then come up as connected.
6. Go back to LLeap Instructor icon
7. Click on Instructor Application
 - o Select Simbaby
 - o Click "OK"
 - o Starting simulator will come up
 - o Go to Manual Mode
 - o Select " Healthy patient", press OK and you will find yourself in the programme....
 - o Start Session
8. To use a patient monitor
 - o Turn on Durabook
 - o Click on LLeap
 - o Ensure you are connected to WiFi
 - o If not, Click on Network Selector, find SimLink network and press OK when prompted
 - o Prompted to allow, select "yes",
 - o then the Network should then come up as connected
9. Click on Patient Monitor
 - o Select Simbaby
 - o Click "OK"

To Shut down

1. Go to the Stop (Square icon as in a Start/Stop button) in LLeap application
2. Click and select "EXIT"
 - a. There will be two options.
 - b. Ensure you are selecting "Exit LLeap and Shut Down " (there will be a tick by 'simulator')
 - c. Press OK