

### INSTRUCTIONS

1. Collect SimTeen and 2 x laptops: Instructor PC and Patient Monitor PC (IBM Tablets).
2. Transfer manikin to bed using PAT slide.
3. Turn on Manikin (switch on right side of manikin). Wait for orange blinking lights on switch panel to stop flashing and turn green. (End up with two orange and one green light on switch panel)
4. Turn on Instructor PC and Patient Monitor PC by pressing the power switch on both. Both can be controlled using touch screen, mouse or touch pad.
5. On Instructor PC.
  - Turn on Instructor Laptop
  - Click on large LLEAP icon to open LLEAP programme
  - Click on LLEAP – Instructor Application icon
  - Select Patient Simulator – SimMan Classic
    - a. Select Manual mode
    - b. Select healthy patient theme, click ok.
    - c. Click Start Session
6. On Patient Monitor Laptop.
  - a) Click on large LLEAP icon on desktop to open LLEAP programme
  - b) Click on Patient Monitor icon and select Patient Simulator – SimMan Classic
  - c) Check that there is sound and display on the patient monitor laptop by selecting ECG, SpO2 on Learner Patient Monitor Buttons Tab.
7. On Instructor PC.
  - Go to sounds and select sound cough
  - Activate patient monitor by clicking on ECG, Sats, this should be displayed and you should hear monitor sounds
8. Auscultate chest
9. Check pulses
10. Make sure the chest is moving and eyes blinking
11. Press the space bar to commence scenario setup. The space bar can be used to toggle the manikin in and out of the standby mode. When in standby mode (the power button on the manikin will change from green to red.) Press the space bar on the instructor laptop to take off standby mode.