

### INSTRUCTIONS

1. Collect Grey Trolley Case with SimNewb and patient monitor tower
2. Take SimNewb, Instructor PC (Laptop), Linkbox out of the drawers
3. Position SimNewb on bed/cot
4. Connect Linkbox to SimNewb. Turn on link box
5. Wait for tone, cough, then two green lights
6. Position Grey Trolley Case with patient monitor tower in place of usual monitor and connect to mains power using orange cable. Turn power on
7. On the Instructor Laptop
  - Connect power, and VGA cable on trolley to laptop
  - Turn on laptop, open LLEAP software on laptop
  - Choose LLEAP instructor application.
  - Select patient simulator and wait for simulator to be identified.
  - Click on SimNewb link box option. (Do not select Virtual Manikin) Monitor will open.
  - Click on run a session.
  - Select a theme (Healthy Patient), click OK
8. Link box light changes to blue
9. Check sound cough
10. Activate monitor tools/patient monitor, wait until connected to monitor approx. 15 seconds
11. Check monitor sound
12. Stop motion and set desired movement
13. Check regulator PSI is set to mark at 1.1 bars
14. Auscultate chest
15. Check pulses
16. Check airway & BMV, intubate

**Commence scenario setup.**

### INSTRUCTIONS

1. Collect grey SimNewB Case and black backpack (SimJunior Backpack)
2. Plug Link box to the cable from the manikin.
3. Turn on link box  
Wait for manikin to turn on and then two green lights on link box
4. Turn on Patient Monitor Laptop / Tower (if you connecting using blue tower)
  - Click on large LLEAP icon to open LLEAP programme
  - Click on Patient Monitor icon on portal and select Patient Simulator – SimNewB
5. On Instructor Laptop.
  - Turn on Instructor Laptop
  - Click on large LLEAP icon to open LLEAP programme
  - Click on LLEAP – Instructor Application icon
  - Select Patient Simulator – SimNewB
    - a. Select Manual mode
    - b. Select healthy patient theme, click ok.
    - c. Click Start Session
    - d. Check that there is sound and display on the patient monitor laptop by selecting ECG, SpO2 on Learner Patient Monitor Buttons Tab.
6. On the Manikin.
  - Check sound
  - Auscultate chest
  - Check pulses
  - Check airway & BMV

**Commence scenario setup**