

# MINIMED® 640G

## BOLUS WIZARD & TEMP BASAL QUICK REFERENCE GUIDES


Medtronic



## DELIVERING A BOLUS AND REVIEWING IT

### Deliver Food and Correction Bolus

1. Test BG. Select **Bolus**. Select **Bolus Wizard**.
2. If using linked meter, BG is on screen. If not, select **BG**. Use  $\uparrow$  or  $\downarrow$  to enter BG and press  $\odot$ .
3. Select **Carbs**.
4. Use  $\uparrow$  to enter grams of carb and press  $\odot$ .
5. Select **Next**.

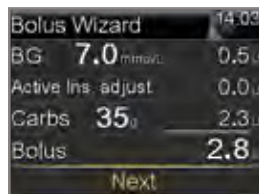


6. Select **Deliver Bolus**.



NOTE: **Active Ins. adjust.** is the active insulin from previous boluses that is adjusted (subtracted) from the correction dose. In this example, there was no active insulin to subtract.

### Deliver Correction Bolus – no food


1. Test BG. Select **Bolus**. Select **Bolus Wizard**.
2. If using linked meter, BG is on screen. If not, select **BG**. Use  $\uparrow$  or  $\downarrow$  to enter BG and press  $\odot$ .
3. Press  $\checkmark$  to **Next**.
4. Select **Deliver Bolus**.

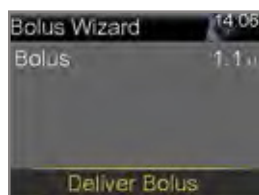

NOTE: In this example, there was active insulin to adjust – it was subtracted from the correction dose.

### Deliver Food Bolus – no correction

1. Select **Bolus**. Select **Bolus Wizard**.
2. Press  $\checkmark$  to **Carbs** and press  $\odot$ . Press  $\uparrow$  to enter grams of carb and press  $\odot$ .
3. Select **Next**.
4. Select **Deliver Bolus**.

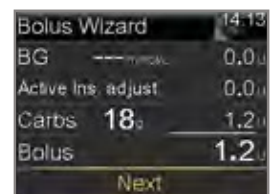

NOTE: Active Insulin is never adjusted (subtracted) from a food bolus.

### Review Bolus History

1. Press  $\text{Ⓜ}$ .
2. Select **History**.
3. Select **Daily History**.
4. Select the day you want to view.
5. You will see bolus deliveries listed in the history.



intermed\*

You. Me. Make a difference.



# STARTING A TEMP BASAL

## Temporary (Temp) Basal Rate

This feature lets you immediately increase or decrease your basal insulin for the period of time (duration) that you set. It is often used for exercise and sick days. A Temp Basal can be set in either Percent (delivers a percent of the current basal rate) or by Rate (delivers the amount that you enter).

### Set Temp Basal Rate

1. From Home Screen, select **Basal**.
2. Select **Temp Basal**.
3. Press to set duration and press .
4. Select **Next**.
5. Select **Percent**.
6. Press or to enter the percent of current basal rate desired.

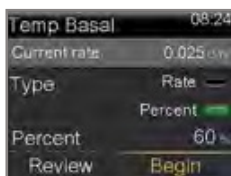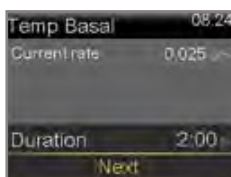

NOTE: If you choose to use Rate, select **Type**, and you can then enter the U/hr desired.


7. Select **Begin**.

NOTE: The Home screen reads Basal (T) since you have a Temp Basal active. Select **Basal (T)** to review the details of the active Temp Basal. When the Temp Basal is complete, the basal will automatically return to the regularly programmed basal rate.

### Cancel Temp Basal Rate

If you ever set a Temp Basal and decide you do not need it, it can be cancelled.

1. From Home Screen, select **Basal (T)**.
2. Select **Cancel Temp Basal**.

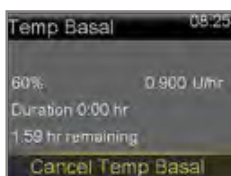

NOTE: Basal rate has now returned to the currently programmed rate.

# STARTING A PRESET TEMP BASAL

## X Starting a Preset Temp Basal

When you are ready to use your Preset Temp Basal:

- 1) From the Home screen, select **Basal**
- 2) Select **Preset Temp**.



Preset Temp now appears on the Basal screen

- 3) Select the Preset Temp you wish to start.



- 4) Select **Begin**.

The Home screen will again show **Basal (T)**.



Proudly Distributed By:

InterMed Medical Limited, 71 Apollo Drive, Albany, Auckland 0632

Toll Free: 0800 333 444 Facsimile: 09 415 9045 Email: sales@intermed.co.nz

24-hour Toll Free: 0800 633 487 For Technical Support

intermed

You. Me. Make a difference.