

## Virtual Setup using LLEAP computer software and ELO monitor

**Note: Manikin is static and tetherless**

1. Turn on instructor PC
2. Check power if battery is low connect to power
3. Position ELO monitor as required
4. Connect ELO to power
5. Connect VGA cable to PC and screw in connections
6. Press power button on side of ELO monitor to turn it on
7. Back to the PC: Double click on the Laerdal Simulator Application icon on desktop
8. Double click on LLEAP instructor application (picture with manikins)
9. Menu opens, select simulator – virtual simulator on top right of screen
10. Double click on local computer icon
11. Select manikin (e.g. Sim Baby)
12. Starting simulator opens and connects.
13. The first time this is selected it will ask for select debriefing system – to bypass this click on advanced options (bottom right)
14. Select continue without debriefing system
15. Select Profile – international
16. Run a session – manual mode
17. Select theme – healthy patient (top left) click on OK
18. Start session
19. On PC if ELO not shown yet click on Tools / patient monitor to activate ELO screen
20. If sound is required for monitor attach audio cable from ELO to PC. Monitor sounds and voice will come from monitor