


## FARRELL™ VALVE SYSTEM

WITH EQUILIBRIUM TECHNOLOGY™



# FARRELL VALVE SYSTEM CLINICAL GUIDE TO SET UP

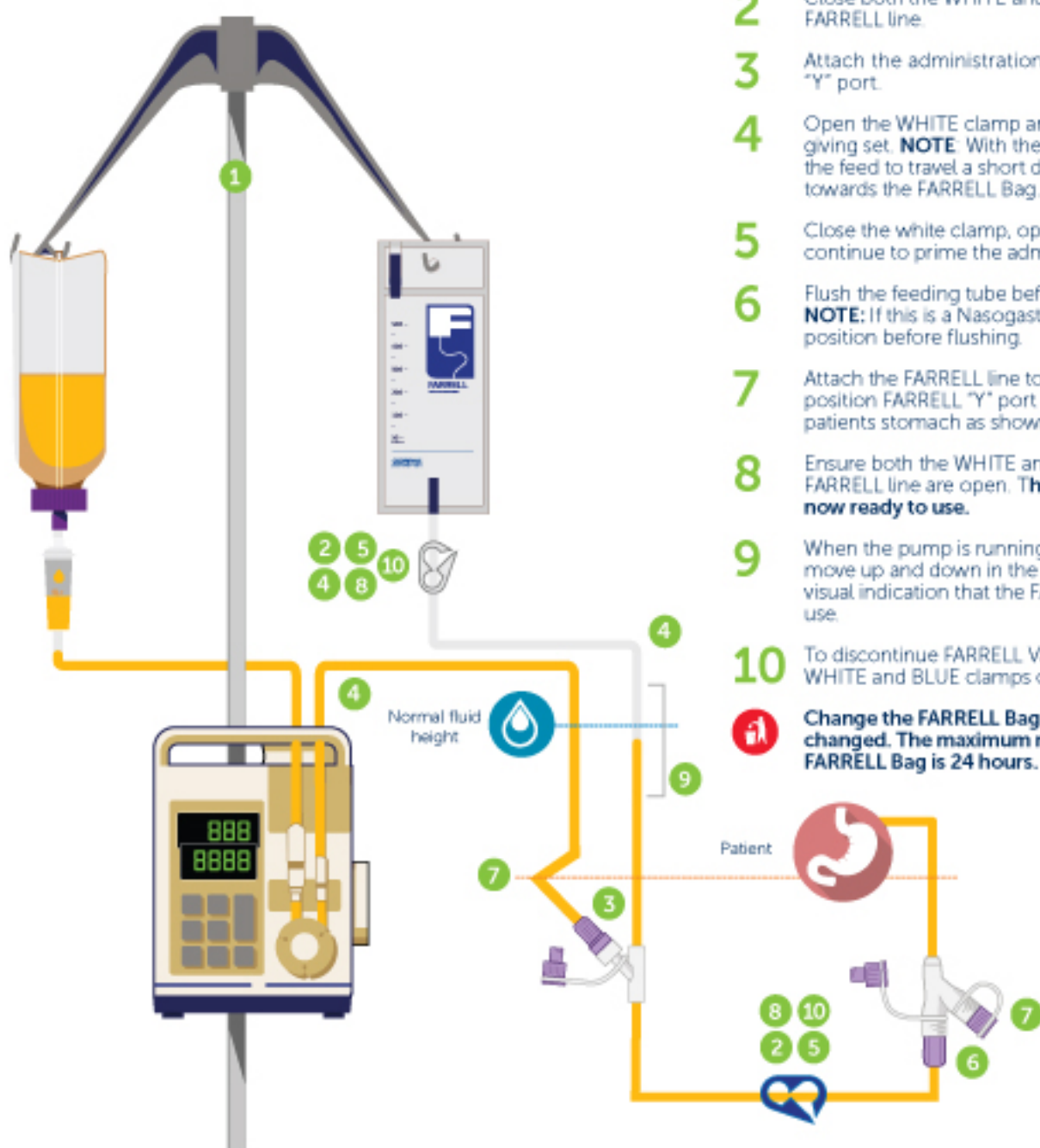

- 1 Hang FARRELL Bag on same I.V. pole as feeding container. **IMPORTANT: THE FARRELL BAG SHOULD BE THE SAME HEIGHT AS THE FEEDING CONTAINER.**
- 2 Close both the WHITE and the BLUE clamps on the FARRELL line.
- 3 Attach the administration/giving set to the FARRELL "Y" port.
- 4 Open the WHITE clamp and prime the administration/giving set. **NOTE:** With the pump in operation, allow the feed to travel a short distance in the 'up line' towards the FARRELL Bag.
- 5 Close the white clamp, open the BLUE clamp and continue to prime the administration/giving set.
- 6 Flush the feeding tube before administering feed. **NOTE:** If this is a Nasogastric tube (NG), confirm tube position before flushing.
- 7 Attach the FARRELL line to the feeding tube and position FARRELL "Y" port below the level of the patient's stomach as shown.
- 8 Ensure both the WHITE and BLUE clamps on the FARRELL line are open. **The FARRELL Valve System is now ready to use.**
- 9 When the pump is running feed will continuously move up and down in the FARRELL Line, providing a visual indication that the FARRELL Valve System is in use.
- 10 To discontinue FARRELL Valve operation, close both WHITE and BLUE clamps on FARRELL Line.

**Change the FARRELL Bag when the feed bag is changed. The maximum recommended use of FARRELL Bag is 24 hours.**