



Safe Care Bulletin

Learning from patient safety events:

The aim of this bulletin is to highlight the changes in practice resulting from reviews of patient safety events with the intention of preventing future similar events.

Grade 3 Hospital Acquired Pressure Injury

We identified that there wasn't an appropriate pressure reducing mattress available for children who weigh less than 25kg and who are at very high risk of pressure injury. An Optima Juve mattress has been purchased and is available via the equipment pool

Optima Juve Mattress



- Connect the colour coded air tubes to the Controller unit and plug into power (will run on battery)
 - Switch on and allow 35-40 minutes for mattress to inflate
 - Place under patient
 - Avoid contact with sharp objects
- *Refer to user manual for full instructions

What to use when to prevent pressure injury

