

Storage of ED Point of Care U/S on PACS

Why store scans on PACS?

- Scans need to be archived for patient documentation
 - Quality improvement - departmental and personal
 - Improves patient care - radiology/specialties can see our scans
 - Useful to review images when reviewing cases (CBD, M&M, peer review) or if there is an adverse outcome
 - Teaching opportunities (showing scans to colleagues, radiology teaching, interesting US review meetings)

What should we store on PACS?

All clinical scans that have been used in patient care.

- Point of care ultrasounds are often difficult and done under pressure, the images are not always perfect, not labelled correctly etc. This is ok.
- All images need to be archived if they have affected patient care.

Process for storage of a Clinical Scan

- Enter patient NHI and last name on scanner
- Scan patient and save labelled images
- End scan
- Scan will automatically be sent to PACS (and be available via Xero on Concerto)
- Complete ROERS request for scan at some point during shift (enables scan to be archived).
- Use CHIPS sign on name in 'clinician performing scan' eg HASLG.

Modality

- X-Ray ACH Adult Emergency/Portables
 - X-Ray ACH General
 - X-Ray Paediatric
 - X-Ray GCC

 - Ultrasound ACH Adult Emergency/Portables
 - Ultrasound Adult General
 - Ultrasound Paediatric
 - Ultrasound NW ACH
 - Ultrasound NW GCC

 - CT ACH Emergency (includes CT Paediatric after hours)
 - CT General
 - CT Paediatric

 - Fluoroscopy/Contrast Study Adult
 - Fluoroscopy/Contrast Study Paediatric

 - Mammography
 - Nuclear Medicine
 - Angio/Interventional
 - MRI Adult
 - MRI Paediatric
 - PET
- (PET referrals require a specific approval process - see 'PET Information' on the Radiology Intranet site)
- Adult POCUS (AED Only)



Please provide some additional information for this item.

Item **Adult POCUS (AED Only)**

Status

Type of Scan

Question to be answered

Clinician performing scan

Save and Close

Cancel Item

-
- Currently you must enter information under Adult POCUS and also tick the adult POCUS box (even though it says AED only)

FAQ

Will my scan be reported?

- No. Point of care scans should be reported by the doctor at the time of the scan and documented in the notes.
- *Will anyone be looking at my scans?*
- We will be auditing clinical scans for quality improvement only.
- *What if I forget to complete a ROERS during that shift?*
- You will need to complete one ASAP, re archive the scan from the machine and email the PACS team (pacs@adhb.govt.nz). If you do not complete a ROERS on shift your scan will be deleted the following day.

Contact

- Any questions/concerns about the process please contact Christine Brabyn SMO
cbrabyn@adhb.govt.nz
- Any issues with sending images or wrongly labelled images during working hours (0800-1630, Mon-Fri) call PACS ext 24500.
- Out of hours email pacs@adhb.govt.nz with the details or call the next working day. • If there is an issue with ROERS call hA service desk as usual ext 27000.